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POLYESTER TAPE, braided, nonabsorbable, undyed or dyed, sterile

- Cervical suture
- Polyester tape (unneeded)

WARNINGS AND PRECAUTIONS

- Read the instructions for use in their entirety before using the product.
- The product should only be used by qualified and trained personnel. It is the physician's responsibility to be familiar with both appropriate surgical techniques and anatomy prior to use.
- The product is delivered sterile (sterilization method: ethylene oxide).
- The product is intended for single use only.
- Do not resterilize or reuse the product as critical changes in performance and safety are possible.
- Do not use after expiration date.
- Do not use if the product shows signs of damage (e.g. cuts or kinks) or if the outer packaging or sterile packaging has been damaged or accidentally opened prior to use.
- Use the product only for the intended indication.
- Please contact the manufacturer for further information.

STERILE EO

DESCRIPTION AND INTENDED PURPOSE

The polyester tape is a synthetic, braided, nonabsorbable tape composed of polyethylene terephthalate (PET). It is supplied dyed with D+C Green, Colour Index Nr. 61565, or undyed.

Polyester tapes are available with various types of needles made of surgical steel for use as cervi-

cal sutures or unneeded. They are available in various lengths.

Cervical sutures are intended for use in the cervical cerclage procedure. The unneeded polyester tapes are used as stay or retracting tapes.

INSTRUCTIONS FOR USE

Armed with a curved needle the polyester tape is inserted beneath the cervical mucosa until it encircles the entire cervix. The tape is then knotted and tightened to close the cervix, yet leaving a slight opening. In order to prevent damage to the needle while suturing it is recommended that the needle be grasped towards the swaged end at a point between the end of the first third and the mid-point.

The unneeded polyester tapes can be used to temporarily retract organs and parts of organs out of the surgical field, thus facilitating the surgeon's work in that area. Afterwards the tapes are removed.

ACTION

PET tapes are nonabsorbable and do not lose their tensile strength in the tissue, even over prolonged periods of time. The cervical suture adequately reinforces closure of the cervix in the presence of premature opening or cervical incompetence during the second trimester of pregnancy. The cerclage is removed at the end of the pregnancy.

The unneeded polyester tapes are suitable for use as slings to retract organs and other structures because their flat form and pliable nature prevents them from cutting into the tissue.

CONTRAINDICATIONS

The device must not be used in patients with allergies to any of the product components.

RESTRICTIONS AND LIMITATIONS OF USE / NOTES / INTERACTIONS

At the end of pregnancy the cervical cerclage should be removed to avoid erosions into neighbouring tissues and organs. The surgical instruments used should be handled with care to

avoid damage to the polyester tape. Bending the needles or grasping them outside the recommended area can result in needle breakage. Needles should be handled and disposed of with particular care due to the risk of infection from needle stick injuries.

The device is not intended specifically for use in direct contact with the heart, the central circulatory system, or the central nervous system.

ADVERSE REACTIONS

The following adverse reactions may occur despite correct use of the polyester tape:

Minimal transient tissue reaction in the form of localized irritation and inflammatory foreign-body reactions; infection may be prolonged when used in infected tissues.

Users and/or patients are encouraged to report any serious incident that has occurred in relation to the device to the manufacturer and the competent authority.

STORAGE CONDITIONS

The storage conditions apply as indicated on the label.

DISPOSAL INFORMATION

Disposal of unused products or waste material with particular care due to the potential risk of contamination and/or injury. In all other cases observe the specific national regulations.

MEANING OF THE SYMBOLS

Symbols are explained at the end of the instructions for use.

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SERAG-WIESSNER GmbH & Co. KG